




Program Mode


1. Press OPTS until PROGRAM MODE is displayed
2. Press ON .
3. Enter the rate for Step 1 and press OK .
4. Enter the volume for Step 1 and press OK .
5. Repeat Steps 3 and 4 until all steps are complete and press OK .
6. Review the Total VTBD and press OK .
7. Press START .

Ramp Taper Mode

1. Press OPTS until RAMP TAPER MODE is displayed
2. Press ON .
3. Enter the total bag volume to be delivered and press OK .
4. Enter the Ramp Up time in hours and minutes and press OK .
5. Enter the Taper Down time in hours and minutes and press OK .
6. Enter the Total Time for the infusion in hours and minutes and press OK .
7. Review the calculated Max Rate and press OK .
8. Press START .

Alarm Troubleshooting

- Air In Line!**
- Check for air in air-in-line detector.
 - Check that tubing is properly inserted in air-in-line detector.
 - Remove air from line.
 - Plug pump into wall outlet.
 - Close door.
- Dwnstream Occlusion!**
- Open clamps.
 - Check for kinked tubing.
 - Check IV site.
 - Check occlusion pressure setting.
- Pump Stopped!**
- Pump is stopped.
 - Either start infusion or press  to extend hold time 3 minutes.

Silence alarms by pressing .

Refer to Instructions for Use for complete operating instructions.

B | BRAUN

B. Braun Medical Inc.
824 Twelfth Avenue
Bethlehem, PA 18018
1-800-227-2862
www.bbraunusa.com

Vista™ basic

Infusion Pump



Quick Reference




B | BRAUN

Rx only
Vista™ and Working Together for Excellence in Safety
are a trademarks of B. Braun Medical Inc.
©2002, B. Braun Medical Inc.


950853-Rev B 06/02

Important Keys

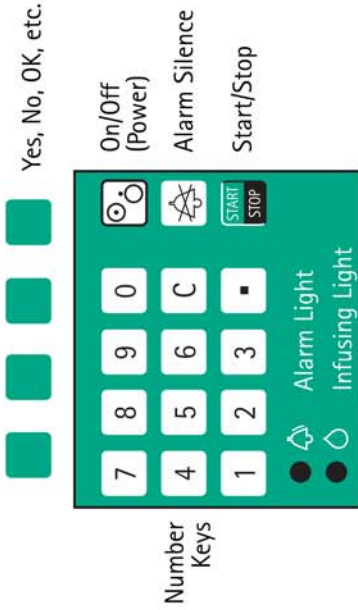

Setting the Pump

1. Press (On/Off).
2. Press button on top of door to open.
3. Insert tubing into pump.
 - Push open tube pincher **A**
 - Place tubing in tube pincher **A**
 - Insert tubing along channel **B**
 - Push tubing into Air In Line Detector **C**

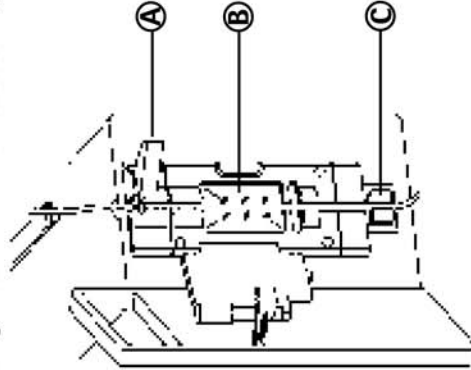

4. Close Door.

Piggyback Mode

1. Press OPTS until PIGGYBACK is displayed
2. Press ON .
3. Press END .
4. Press PIGY .
5. Enter Rate/VTBD or VTBD/Time as in Continuous Mode.
6. Press START .

5. Is this a new therapy?
 - Yes : clears current infusion data.
 - No : returns pump to last programmed infusion.
6. Prime set?
 - Yes : follow prompts to prime tubing.
 - No : begin programming infusion.

Continuous Mode


1. Enter the rate using the number keys.



2. Press VTBD .
3. Enter the bag volume using the number keys.
4. Press OK .



5. Press START .

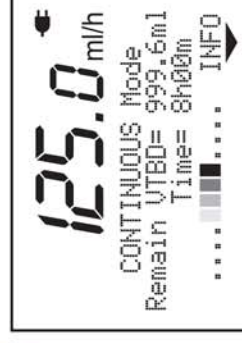

Changing the Rate

1. While infusing, enter the new rate using the number keys.
2. Press OK .



NOTE:

If the OK key is not pressed within 10 seconds of the change, the infusion will continue at the original rate.

Changing the VTBD

1. Stop infusion.
2. Press VTBD .
3. Press **C**.
4. Enter the new volume to be delivered.
5. Press OK .
6. Press .

